
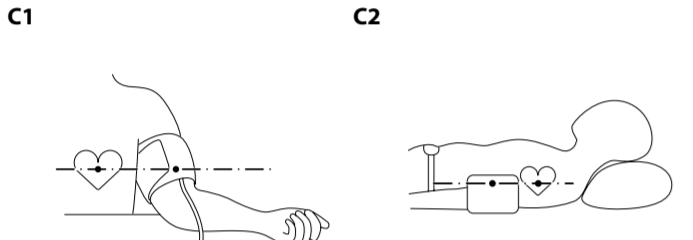
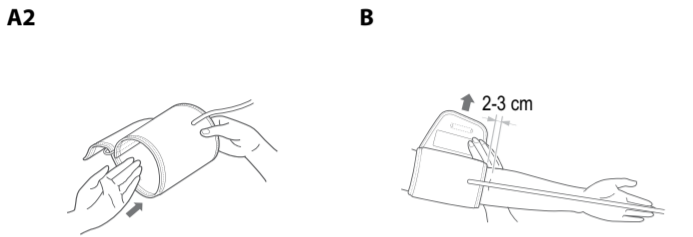
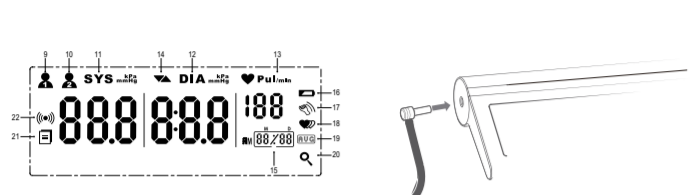
KONIG

KN-BLDPRESS40B / LS805-B

Blood pressure monitor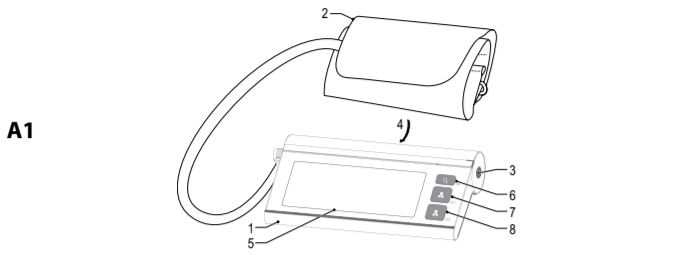

English

Description (fig. A)
The blood pressure monitor is a device used to measure a person's blood pressure. The device measures the systolic blood pressure, the diastolic blood pressure and the pulse rate of an adult individual.

1. Monitor	
2. Cuff	
3. DC input	<ul style="list-style-type: none">Connect the power adapter to the DC input. Insert the mains plug of the power adapter into the wall socket.
4. Battery compartment	<ul style="list-style-type: none">Open the battery compartment. Insert the batteries (4x AAA) into the battery compartment. Close the battery compartment.
5. Display	
6. Memory button	<ul style="list-style-type: none">Press the button to read measurements from the memory. Clock mode: Press the button to set the date and the time.
7. User 1 button	<ul style="list-style-type: none">User 1: Press the button to switch on or off the device. Clock mode: Press the button to confirm the date and the time.
8. User 2 button	<ul style="list-style-type: none">User 2: Press the button to switch on or off the device. When the device is switched off, press and hold the button to pair the blood pressure monitor with your mobile device.
9. User 1 indicator	<ul style="list-style-type: none">The indicator comes on when the device is switched on using the User 1 button.
10. User 2 indicator	<ul style="list-style-type: none">The indicator comes on when the device is switched on using the User 2 button.
11. Systolic blood pressure indicator	<ul style="list-style-type: none">The indicator shows the systolic blood pressure.
12. Diastolic blood pressure indicator	<ul style="list-style-type: none">The indicator shows the diastolic blood pressure.
13. Pulse rate indicator	<ul style="list-style-type: none">The indicator shows the pulse rate.
14. Inflating/deflating indicator	<ul style="list-style-type: none">The indicator shows when the device is inflating (arrow up) or deflating (arrow down).
15. Date indicator	<ul style="list-style-type: none">The indicator alternately shows the date and the time.
16. Low-battery indicator	<ul style="list-style-type: none">The indicator comes on when the batteries are low.
17. Movement detection indicator	<ul style="list-style-type: none">The indicator comes on when the device detects movement.
18. Irregular-heartbeat indicator	<ul style="list-style-type: none">The indicator comes on when the device detects an irregular heartbeat.
19. Average value indicator	<ul style="list-style-type: none">The indicator shows the average value of the last three measurements.
20. Memory indicator	<ul style="list-style-type: none">The indicator comes on when the device is in memory mode.
21. Pending-data indicator	<ul style="list-style-type: none">The indicator indicates that data are pending to be transmitted to your mobile device.
22. Transmitting-data indicator	<ul style="list-style-type: none">The indicator indicates that data are being transmitted to your mobile device.

Definities

- Systolic blood pressure: maximum arterial pressure during contraction of the left ventricle of the heart.
- Diastolic blood pressure: minimum arterial pressure during relaxation and dilatation of the ventricles of the heart when the ventricles fill with blood
- Pulse rate: number of pulsations in the radial artery per minute.
- Hypotension: low blood pressure.
- Prehypertension: precursor to chronic high blood pressure.
- Hypertension: high blood pressure.
- Hypertensive emergency: high blood pressure with acute impairment of one or more organ systems which can result in irreversible organ damage.

Blood pressure classification

Blood pressure classification	Systolic blood pressure (mmHg)	Diastolic blood pressure (mmHg)
Hypotension	< 120	< 80
Normal	120 - 129	80 - 84
Prehypertension	130 - 139	85 - 89
Hypertension (stage 1)	140 - 159	90 - 99
Hypertension (stage 2)	160 - 179	100 - 109
Hypertensive emergency	≥ 180	≥ 110

Safety

- Read the manual carefully before use. Keep the manual for future reference.
- Only use the device for its intended purposes. Do not use the device for other purposes than described in the manual.
- Do not use the device if any part is damaged or defective. If the device is damaged or defective, replace the device immediately.
- The device is designed for adults. Do not use the device on infants or young children. Before use on older children, consult your physician.
- Do not use the device if you suffer from arrhythmia (irregular heartbeats).
- If you are in doubt whether to use the device or not, consult your physician.

- The measurements are only an indication of your blood pressure level. If the device measures a high or low blood pressure level, consult your physician.
- Only use the supplied cuff.
- Measure your blood pressure at approximately the same time each day.
- Before measurement, stay calm and rest for 5 minutes. Relax at least 1 minute between multiple measurements to allow the blood circulation to recover. Too frequent measurements can cause injury due to blood flow interference.
- Do not speak or move during measurement.
- Do not expose the device to water or moisture.
- Do not use the device in areas where interference from other devices can cause inaccurate measurements.
- Do not use the device in a moving vehicle.

Battery safety

- Use only the batteries mentioned in the manual.
- Do not use old and new batteries together.
- Do not use batteries of different types or brands.
- Do not install batteries in reverse polarity.
- Do not short-circuit or disassemble the batteries.
- Do not expose the batteries to water.
- Do not expose the batteries to fire or excessive heat.
- Batteries are prone to leakage when fully discharged. To avoid damage to the product, remove the batteries when leaving the product unattended for longer periods of time.
- If liquid from the batteries comes into contact with skin or clothing, immediately rinse with fresh water.

Installation

- Download and install the "Transhealth" app from Apple App Store or Google Play Store on your mobile device.
- Open the "Transhealth" app.
- Activate Bluetooth on your mobile device.
- Go to the settings menu
- Select "Add device" in the settings menu.
- Press and hold the User 2 button for 5 seconds. The blood pressure monitor will be paired with your mobile device.

Use

Setting the date and the time

- When the device is switched off, press and hold the User 1 button for three seconds. The device is in clock mode.
- Press the memory button to set the hour. Press the User 1 button to confirm.
- Press the memory button to set the minutes. Press the User 1 button to confirm.
- Press the memory button to set the month. Press the User 1 button to confirm.
- Press the memory button to set the day. Press the User 1 button to confirm.
- Press the memory button to set the year. Press the User 1 button to confirm.

Attaching the cuff to the monitor (fig. B)

- Refer to the illustration to attach the cuff to the monitor.

Applying the cuff (fig. C)

- Refer to the illustration to apply the cuff.

Body posture during measurement (fig. D)

- Refer to the illustration for the correct body posture during measurement.

Measuring the blood pressure

- To switch on the device, press the start/stop button. The display shows all display items.
- The monitor starts to seek zero pressure.
- The monitor inflates the cuff until there is sufficient cuff pressure.
- The monitor slowly releases air from the cuff and carries out the measurement. The display shows the systolic blood pressure, the diastolic blood pressure and the pulse rate. The results are stored in the memory automatically.
- To switch off the device, press the start/stop button again.

Reading measurements from the memory

- Press the memory button. The display shows the average value of the last three measurements.
- Repeatedly press the memory button to navigate through the results.

Deleting measurements from the memory

- When the display shows a result, press and hold the memory button for 3 seconds. All results in the memory are deleted after three beeps.

Error code	Cause	Solution
E1	Data communication error.	Make sure that Bluetooth is activated on your mobile device. Make sure that your mobile device is within the Bluetooth range.
E3	The cuff is too loose during inflation.	Apply the cuff correctly and measure again.
E10 / E11	The device has detected motion during measurement.	Relax for a moment and measure again after 5 minutes.
E20	The device does not detect the pulse signal.	Loosen the clothing on the arm and measure again.
E21	Measurement error.	Relax for a moment and measure again after 5 minutes.
EExx	Calibration error.	Measure again after the error code. If the display still shows the error code, contact the local distributor or the factory.

Cleaning and maintenance

Warning!

- Do not use cleaning solvents or abrasives.
- Do not clean the inside of the device.
- Do not attempt to repair the device. If the device does not operate correctly, replace it with a new device.

- Clean the outside of the device using a soft, damp cloth.
- If you use the device frequently, disinfect the cuff at least twice a week. Apply ethyl alcohol (75-90%) to a soft cloth, wipe the inner side of the cuff with the cloth and allow the cuff to dry naturally.

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