
FERVOR

BLOOD PRESSURE MONITOR

MANUAL

CE 0123

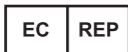Share Info GmbH
Heerdter Lohweg 83, 40549
Dusseldorf, Germany

Guangdong Xinyu Electronic and Technology Co.,Ltd.
No. 42, CuiCheng Road, Cuiheng New District, 528400
Zhongshan, Guangdong, PEOPLE'S REPUBLIC OF CHINA

ECB009

BLOOD PRESSURE MONITOR

WHAT IS BLOOD PRESSURE?


The blood pressure is the pressure the blood is exerting on the walls of the arteries. The blood pressure in the arteries is subject to continuous variation during a heart cycle. The highest pressure within this cycle is called 'systolic pressure' (SYS), the lowest pressure is called 'diastolic pressure' (DIA). With these two blood pressure values, a doctor can determine a patient's blood pressure. Your blood pressure may be influenced by many different factors as physical effort, anxiety or the time of the day etc. Generally, blood pressure is low in the morning and raises in the afternoon towards the evening. In summer, blood pressure is lower than in winter.

WHY IS IT A GOOD THING TO MEASURE BLOOD PRESSURE AT HOME

Having one's blood pressure measured by a doctor in a hospital or clinic environment tends to stimulate nervousness in a patient and may lead to a higher blood pressure. Blood pressure varies in accordance with a variety of conditions and so judgment is not always possible on the basis of a single measurement. Blood pressure measured first thing in the morning after getting up, before eating, and keeping still, is known as the fundamental blood pressure. In practice it is rather difficult to record the fundamental blood pressure but measurement at home in the right environment aids this.


VARIATIONS OF BLOOD PRESSURE

Individual blood pressure is subject to variations on a day-to-day basis or in relation to the time of the year. These variations are more perceptible in patients with high blood pressure. Usually, blood pressure is higher during work and is lowest in the night when sleeping. The figure shows the variations of the blood pressure in the course of a day with measurements taken every 5 minutes. The bold line represents the sleep phase. The increase in blood pressure at 4 pm (shown as letter 'A' in the figure) and at midnight (shown as letter 'B' in the figure) mark pain events of the patient.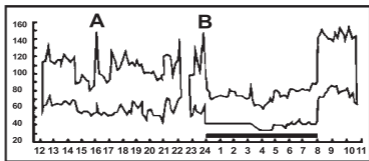

BLOOD PRESSURE CLASSIFICATION OF THE WHO

The standards for the determination of high or low blood pressure independently from the patient's age – as shown on the table – were determined by the World Health Organization (WHO).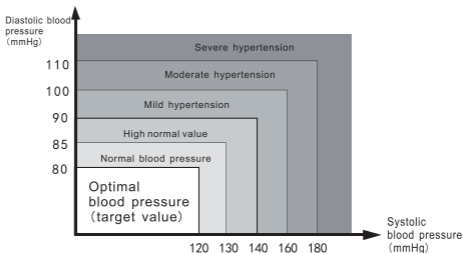

IMPORTANT INSTRUCTIONS OF THE DEVICE

• Blood pressure should be judged by a medical doctor or other medical specialist who is familiar with your case history. Through the regular use of the blood pressure gauge you can continuously record your pressure values. These records can be used for therapeutic purposes.

- Tightly wrap the cuff around your upper arm. The cuff must be positioned on the level of the heart.
- The monitor must not be moved or shaken during measuring as this would lead to inaccurate results.
- Carry out measuring in a still and relaxed position.
- Do not wrap the cuff around the sleeve of a jacket, shirt which could make measuring impossible.
- Keep in mind that blood pressure naturally varies in the course of the day and is influenced by many different factors as, e.g., smoking, alcohol, drugs and physical effort. We recommend not to smoke, eat or do physical exercises for about 30 minutes before measuring.
- To the people with following condition which leads to circulatory disorders (diabetes, kidney diseases, hardening of the arteries), the values measured with this device may be lower than those measured with a gauge taking the blood pressure at the upper arm. Consult your doctor to make sure that measuring blood pressure at your upper arm leads to accurate measuring values.
- The measuring device is designed for the measuring of blood pressure in adults only.
- Do not use the device if your upper arm is injured.
- If the automatic inflation function of the cuff does not stop automatically it must be opened and taken off immediately!



Note: This device is not suitable for invasive blood pressure measuring in adults or children. It may only be used for non-invasive blood pressure monitoring and only in adults!

PARTS DESCRIPTION

- Cuff jack
- Type-C jack
- LCD display
- SET button
- MEM button
- ON/OFF button

THE SYMBOLS ON THE LCD DISPLAY

- Systolic blood pressure
- Diastolic blood pressure
- WHO classification symbol (blood pressure level)
- Pulse display
- Low battery symbol
- Date/Time display
- Average value symbol (AVG)
- Memory symbol
- User A
- User B
- Irregular heartbeat symbol
- Heartbeat symbol (flashes during measurement)
- Number of measurement memory
- Unit (mmHg/kPa)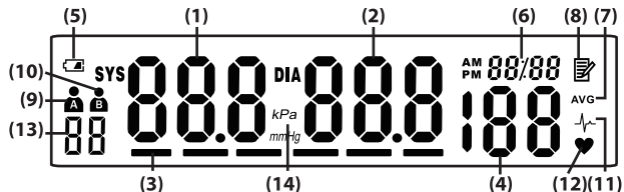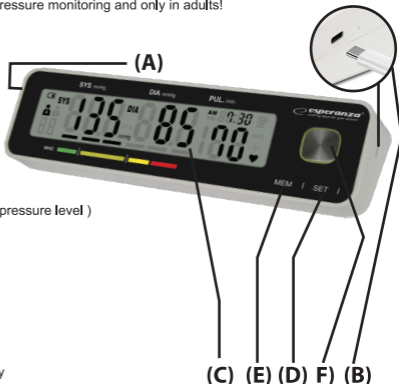

USING THIS DEVICE FOR THE FIRST TIME

Selecting the power source

You can optionally operate the device by a AC adapter with type C connector (output : DC, 5V, 1A) or 1.5V "AAA" alkaline batteries. Note : AC adapter is not included in the set,

Using the AC adapter

- Use only the CE approved AC adapter to avoid damaging the device.
- Ensure that the AC adapter and cable are not damaged.
- Plug the adapter cable into the AC adapter jack on the right side of the device.
- Plug the adapter into your mains outlet. When the AC adapter is connected, no battery current is consumed.

Battery Installation

- Lift the hook on the bottom of the battery cover and lift the cover off in the direction of the arrow.
- Install 4 "AAA" size batteries so the + (positive) and - (negative) polarities match the polarities of the battery compartment, replace the battery cover. Make sure that the battery cover is securely in position.

Battery replacement

Low Battery Indicator

- When the Low Battery Indicator appears on the display, turn the device off and remove all the batteries. Replace with 4 new batteries at the same time. Long-life alkaline batteries are recommended.
- To prevent the damage of device from leaked battery fluid, please take out battery if the device unused in a long time (generally more than 3 months). If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Contact a physician immediately.
- Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.

MEASURING INSTRUCTIONS


Set the date, time and measurement unit

- Before taking measurements, set the current date, time and measurement unit. The measured value will be stored in memory together with the date and time of measurement.
- Press ON/OFF button to switch on the device.
- Press the button SET, the default year will flash on the display. During the initial setup the display will show default year, month and day.
- Set the current year by pressing the button MEM and confirm it by pressing button SET. And the display show another page with date and time.
- The first two numbers (which means month) are flashing, Press the button MEM to put the right month and confirm by pressing button SET.
- The date numbers flash. Press the button MEM to put the right date and confirm by pressing button SET.
- The hour number flash and press the button MEM to put the right hour, This device is set with 24-hour time format, And confirm the hour with button SET.
- The minute numbers flash. Press button MEM to put the right minute and confirm by pressing button SET.
- The unit number and the letter "mmHg" flashes. Press MEM to select the measurement unit mmHg or kPa. Confirm the unit by pressing button SET. After all settings are done, press ON/OFF button and data are saved.

MEASURING BLOOD PRESSURE

Basic instruction for achieving the most accurate measuring results

- Always take measurements at the same time of day, ideally in the morning, at noon and in the evening under the same conditions or according to the recommendations of your doctor.
- Do not perform measurement sooner than 30-45 minutes after consuming coffee, tea or smoking a cigarette.
- Wait at least 20 minutes after taking a hot shower or bath.
- During measurement sit calmly, relax and don't talk. Do not move the arm to which the cuff is attached.
- Wait approximately 4-5 minutes before measuring again.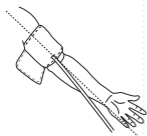

Wrap the cuff around the upper arm

- Open the Velcro fastener and pull to open the cuff.
- Lead your arm through the cuff and pull the cuff up to a position 1-2 cm above your elbow. The cuff is suitable for a perimeter of the upper arm about 22-32 cm.
- Pull the cuff tight around the upper arm. Do not pull on the cuff with exaggerated force.
- Do not pull the cuff too tight, leave a gap of about the width of a finger.

Correct body position for measuring

To receive best possible measuring results make sure that the cuff is held on the same level as your heart. If the cuff is held in a position above or below the heart this may lead to deviations from the exact value.

MEASURING BLOOD PRESSURE IN SITTING POSITION

- Rest your elbow on a table or other surface (as, e.g., a bag).
- Position your arm on the rest in a way that makes sure that the cuff is on the same level with your heart.
- Relax your arm. The palm of your hand must point upwards.


MEASURING THE BLOOD PRESSURE

- Press ON/OFF button to switch on the device. The selected user ID will flash. Then press again ON/OFF button, the pump begins to inflate the cuff. The increasing pressure is continually displayed.
- After automatically reaching an individual pressure, the pump stops and the pressure slowly falls. The pressure value is displayed during the measurement.
- When the device has detected your pulse, the heart symbol in the display begins to blink.
- When the measurement has been concluded, the measured systolic and diastolic blood pressure values, as well as the pulse value will be displayed. When irregular heartbeat rhythm is detected two or more times during measurement, the irregular heartbeat symbol will display after the measurement together with pulse rate.
- The measurement results are displayed until you switch the device off. If no button is pressed for 180 seconds, the device switches off automatically.

IRREGULAR HEARTBEAT DETECTOR

An irregular heartbeat rhythm is defined as a rhythm that varies by 25% higher or lower than average rhythm detected while the device is measuring the systolic and diastolic blood pressure.

The irregular heartbeat symbol displaying on LCD, which indicates that certain pulse irregularities were detected during the measurement. When irregular heartbeat is detected, the symbol will display after the measurement. If the symbol appears more frequently (e.g. several times per week on measurements performed daily) or if it suddenly appears more often than usual, we recommend the patient to seek medical advice. The instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

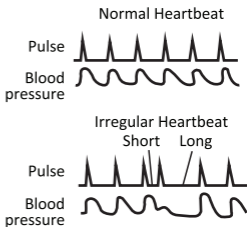SWITCH THE USER (A and B)

User A is set as default user.

In the off mode, press ON/OFF button and hold it for 6 seconds, the icon of user B will show up.

SAVE THE MEASURING DATA AND SHOW THE MEMORY

a. Save data

- After each measurement of the blood pressure measurement, the data of the systolic/diastolic pressures and heartbeat rate of the measuring cycle are automatically saved. The last 99 measuring results can be retrieved from the device's memory. If the number of measuring results exceeds 99, the oldest data in the memory are automatically deleted.
- Each user has 99 memories of measurements. In total there are 2x99 measurement results are saved.

b. Display the stored measurement results

Press ON/OFF button to switch on device. Press the MEM button to enter Into memory mode. The monitor will display the current measured user ID and an average value (AVG) of the last 3 times measurements stored in the device. If measurements are less than 3 sets, it directly displays the latest value. Each time you press the MEM button, it will display the memory value from the latest to the oldest in turn.

c. Delete the stored measurement results

Press ON/OFF button to switch on device. Press the MEM button to enter Into memory mode, press and hold MEM button around 3 seconds, all measurement results are deleted.

d. Delete all stored data and resume to factory settings

Press ON/OFF button to switch on device. Press the MEM button to enter Into memory mode, press and hold both MEM and SET buttons around 3 seconds, all measurement results are deleted. The device resume to factory settings.

MAINTENANCE

- To clean the housing of the device use a cloth moistened with water or a soft cleaning agent, then dry it with a dry cloth. Use a dry cloth to clean the cuff if it should be dirty.
- Do not immerse the device in water.
- Do not use gas or strong cleaning agents.
- If the measuring device is not used for a longer time, the batteries should be removed. (Leaking batteries may damage the device.)

STORAGE

- Protect the measuring device with its accessories from direct sunlight, high temperatures, humidity and dust.
- Do not expose the device to extremely low (below -20°C (-4.0°F)) or high (above 65°C (122.0°F)) temperatures.

ERROR CODES AND SOLUTIONS

Error Code	What Problem Can Be	Diagnosis And Solution
Er2	Measurement error	Please rest for 2 minutes, adjust the cuff, keep quiet during the measurement, do not move or talk.
Er10	Static pressure exceeds the upper limit protection (295mmHg)	Measure blood pressure again
Er4	Zeroing timeout	1. Keep the cuff still when zeroing;
		2. Check the power supply part of the sensor;
		3. Replace the sensor;
Er30	Cuff wearing abnormality	Adjust the cuff to a tightness that can just fit two fingers, and keep the cuff tracheal opening 2cm above the elbow.
Lo	Battery insufficient	Replace the battery

TECHNICAL SPECIFICATIONS

LCD Display size: 147 * 29 mm

Product size: 204 (W) *54 (L) * 40 (H) mm

Operating temperature: 5°C to 40°C

Operating humidity: 15% to 80% RH

Storage and shipping temperature: -25°C to 55°C

Storage and shipping humidity: 10% - 93%RH

Atmospheric pressure range: 80kPa ~ 105kPa

Measuring method: Oscillometric

Pressure sensor: Resistive

Measuring range: DIA: 35-190mmHg; SYS: 60-250mmHg

Pulse: 40 to 180 per minute

Cuff pressure display range: <295mmHg

Memory: Automatically stores 99 measurements for 2 users (total 198 measurements)

Measuring resolution: 1 mmHg

Accuracy: Pressure within ± 3 mmHg / pulse ± 3 % of the reading

Power source A: 4*AAA batteries, 1.5 V (not included)

Power source B: AC adapter (not included): Input: 220-240V, 50/60 Hz. Output: 5V DC 1A

USB cable: Input port-type C output port-USB 80cm

Cuff: 22-32cm

Automatically power off: 180 seconds without operation

WARRANTY

1. This warranty is issued by EDC Poterek Sp. Jawna with address:
ul. Poznańska 129/133, 05-850 Ożarów Mazowiecki, Phone: 022-7213571 w.10 – called hereinafter Esperanza.
2. This warranty confers upon you specific rights and remedies against EDC Poterek Sp. Jawna. Please read this document carefully, by making a claim under this warranty, you will be deemed to have understood and accept its terms.
3. Esperanza warrants that any Esperanza product together with attached accessories is free from defects in materials and workmanship under conditions of its normal use.
4. If you make a claim under this warranty, Esperanza will, at its option, repair or replace any defective product into new one by using new or use factory-refurbished parts. In case of using factory-refurbished parts they will be equal to new products.
5. Defective product should be returned together with receipt, commercial invoice or any other proof of purchase to the place of purchase. It should include all accessories attached in original packing. Esperanza has right to reject warranty if any accessories will be missed.
6. EDC Poterek Sp. Jawna covers the costs of delivery of the goods complained about to the place indicated in the guarantee.
7. EDC Poterek Sp. Jawna is committed to respond to the complaint within 14 calendar days of its notification in the point of sale. EDC Poterek Sp. Jawna will repair the defective product in the shortest possible time. If will be need to import from abroad of any parts which are necessary to repair this time may be extended.
8. If in the performance of its duties EDC Poterek Sp. Jawna provided the entitled person with a warranty instead of a defective item free from defects or made significant repairs to the item covered by the warranty, the warranty period runs anew from the moment of delivery of the item free from defects or returning the repaired item. If EDC Poterek Sp. Jawna mentioned a part of things, the above provision shall apply accordingly to the part mentioned. In other cases, the warranty period is extended by the time during which, as a result of a defect in the item covered by the warranty, the holder of the warranty could not use it.
9. In case of replacing defective product onto new one, Esperanza has right to replace it with newer model with the same technical parameters if the old one model was replaced by new one during warranty period.
10. If you make claim under this warranty, you are deemed to have agreed that repair and (upon availability) replacement, as applicable, under the warranty services described herein, is your sole and exclusive remedy vis-a-vis Esperanza.
11. The warranty is valid in Poland.

EXCLUSIONS:

This warranty does not extend to:

1. Any product not manufactured by EDC Poterek Sp. Jawna
2. Any product that has been damaged or rendered defective as a result of:
 - use it in other way than normal intended use
 - service or repair by anyone other than EDC Poterek Sp. Jawna service
 - improper store, transportation or packing
 - improper installation of product or software
 - mechanical, physical, chemical etc. damages
3. In no event shall EDC Poterek Sp. Jawna be liable for any damages, loss of business, profits, data or loss based on use EDC Poterek Sp. Jawna products with other devices.
4. Products with removed warranty and other labels and products spent or have hard use marks.

All products are covered by EDC Poterek Sp. Jawna 2-year manufacturer's warranty while product is purchased by the consumer (the consumer is a person who buys product for the purpose of not professional or commercial activity). In the case of purchase for the company (for purposes relating to the activities of professional or commercial) warranty is 1 year from date of purchase. The batteries installed in the devices are covered by a 6-month manufacturer's warranty. EDC Poterek Sp. Jawna products have all the necessary safety certificates required in the EU.

According to Directive 2012/19/EU, disposal of waste electrical and electronic equipment together with household waste is prohibited. Their components need to be recycled or disposed separately. Otherwise, they can cause the release of hazardous and noxious substances which pose a threat to the environment and human health. The law requires each consumer to return unnecessary or waste electrical and electronic equipment to the producers, sales agents or waste treatment facilities. Detailed conditions are governed by national legislation. The above symbol placed on the product, in the user manual or on the packaging refers to this requirement. Thanks to proper segregation and compliance with the provisions relating to waste disposal each user contributes significantly to environmental protection.

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Product Service

EU Quality Management System Certificate (MDR)

Pursuant to Regulation (EU) 2017/745 on Medical Devices, Annex IX Chapters I and III
(Class IIa and Class IIb Devices)

No. G10 091719

Manufacturer:

Guangdong Xinyu Electronic and Technology Co., Ltd.

Number 42, Cui Cheng Road, Cuiheng New District
528400 Zhongshan, Guangdong
PEOPLE'S REPUBLIC OF CHINA

SRN Manufacturer - CN-MF-000025849

The Certification Body of TÜV SÜD Product Service GmbH certifies that the manufacturer has established, documented and implemented a quality management system as described in Article 10 (9) of the Regulation (EU) 2017/745 on medical devices. Details on device categories covered by the quality management system are described on the following page(s). The Report referenced below summarises the result of the assessment and includes reference to relevant CS, harmonized standards and test reports. The conformity assessment has been carried out according to Annex IX Chapter I and III of this regulation with a positive result.

The quality management system assessment was accompanied by the assessment of technical documentation for devices selected on a representative basis.

The certified quality management system is subject to periodical surveillance by TÜV SÜD Product Service GmbH. The surveillance assessment shall also include an assessment of the technical documentation for the device or devices concerned on the basis of further representative samples. All applicable requirements of the Testing, Certification, Validation and Verification Regulations TÜV SÜD Group have to be complied with.

For details and certificate validity see: www.tuvsud.com/ps-cert?q=cert:G10_091719

Report No.: GZ2349601

Valid from: 2024-10-29

Valid until: 2029-10-28

Issue date: [2024-10-29]

Christoph Dicks
Head of Certification/Notified Body

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Product Service

EU Quality Management System Certificate (MDR)

Pursuant to Regulation (EU) 2017/745 on Medical Devices, Annex IX Chapters I and III
(Class IIa and Class IIb Devices)

No. G10 091719

Classification: Class IIa
Device Group: Z1203020302 - NON-INVASIVE BLOOD PRESSURE
MONITORING INSTRUMENTS

Intended Purpose: -

The validity of this certificate depends on conditions and/or is limited to the following: - none -

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We, the manufacturer/importer (My, producent/importer):

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Herewith, Declares that the product:

Product description: Blood Pressure Monitor

Model Number: ECB009 as factory model XY-B05

Manufacturer: Guangdong Xinyu Electronic and Technology Co.,Ltd.

**No. 42, CuiCheng Road, Cuiheng New District, 528400 Zhongshan, Guangdong, PEOPLE'S
REPUBLIC OF CHINA**

The products is compatible with the following norms/standards:

Directive (EU) 2017/745 on Medical Devices (MDR), Annex IX, in class IIa

Issued by TUV SUD Product Service GmbH, Notified body: 0123, EC Certificate No.: G10 091719

This certificate is valid from 4th,March 2024 until 3rd,March 2029.

EC Representative : Share Info GmbH

Heerdter Lohweg 83, 40549 Dusseldorf, Germany

The product is inline with regulations listed below:

Medical directive: (EU) 2017/745 on Medical Devices (MDR)

This Declaration of Conformity is issued under the sole responsibility of the manufacturer.

MANUFACTURER

Guangdong, China